

Procedure to Access Excel Reports thru USB for LifeWindow.

- 1- Connect one keyboard to the LifeWindow USB or PS2 port.
- 2- With the LifeWindow software running, press the "**windows**" Key in the keyboard.



- 3- In case you don't have a keyboard, stop the LifeWindow software by pressing the **Freeze**, the **NIBP START** dedicated buttons and simultaneously clicking the **Traces** menu. The LifeWindow software will stop and the desktop screen will be displayed.
- 4- Click in **Start, Settings, Documents** and **My Documents**.
- 5- Click in **Folders, My Computer** and click in **LIFEWINDOW (C:)**.
- 6- Double click in **Patient Data**. This is the default location where the excel reports are saved.
- 7- Click once in the report you want to copy and it will become blue (selected).
- 8- Click in **Edit** and click in **Copy**.
- 9- Connect the USB flash in the USB LifeWindow port.
- 10- The USB drive **(D:)** screen will open. Click in **Open folder to view files**. If the **(D:)** did not open automatically, click in the **(D:)** to open.
- 11- Click in **Edit** and click in **Paste**.
- 12- Close the **Explorer** screen in the LifeWindow.
- 13- Remove the USB drive with the PDF report and connect in the USB port of the PC you want to transfer the file.
- 14- If the LifeWindow software was stopped, click on **Start, Shut Down**, select **Restart** and click **OK**.