

Procedure to Adjust LifeWindow PDF Report Size.

- 1- Connect one keyboard to the LifeWindow USB port.
- 2- With the LifeWindow software running, press the "**windows**" Key in the keyboard.



- 3- Click in **Start, Settings** and **Printers and Faxes**.
- 4- Click one time in **PrimoPDF**, click in **File** and click in **Printing Preferences**.
- 5- Click in **Advanced**. Click in the **Down Arrow** and click in **PostScript Custom Page Size**.
- 6- Using the keyboard, change the **Width:** to 9.00 and the **Height:** to 12.0.
- 7- Click **OK** three times to save the new settings and close the **Printers and Faxes** screen.
- 8- The LifeWindow screen will be displayed and the procedure is finished.