

## Procedure to Update LifeWindow Software

### Items required for this installation:

- All software components for updating (located in another computer in the Network ):

LifeWindow Software Update File “LW6000 Model Version.exe”.

**Note:** Those files are contained in an auto extractable ZIP file with the form LW6000 (Model) (Version). exe ( Ex: LW6000 ERSFCN2P2T V2.99.exe). This executable is available from Digicare and will be sent to your email by request. Please have the serial number for your LifeWindow handy when you call to order the newest software revision for your monitor.

- LifeWindow 6000 connected to a network. Please refer to “**Procedure to Place the LifeWindow 6000 in a Network**”. Procedure #100.
- PC Keyboard (Note: earlier versions of the LifeWindow 6000 had a DIN connector for the PC keyboard ( S/N below V954349), this requires an adapter to convert the keyboard connector PS/2, most commonly used in later PCs, to the old style DIN connector). This adapter is available in computer stores like Compusa and Radio Shack )

### Procedures:

1. Turn the LifeWindow monitor off.
2. Connect the keyboard to the LifeWindow keyboard port in the rear of the LifeWindow.
3. Confirm that the LifeWindow is connected to your Network.  
Please refer to “**Procedure to place the LifeWindow 6000 in a Network**”. Procedure #100.
4. Turn on the LifeWindow.
5. With the LifeWindow on and running the LifeWindow software, position the mouse cursor arrow on top of the Icon “**TRACES**” in the LifeWindow screen task bar. Now with one finger press the gray key “**FREEZE**” in the direct key’s panel and you should hear one beep, keep it pressing, now with another finger press the green key “**NIBP START**” and hear another beep, keep both keys pressed. Only two beeps can be heard until now. Now press with another finger the blue mouse button on top of the unit. The LifeWindow software will stop running and you will get a green screen.
6. Press “**CTRL+ALT+DEL**” in the PC keyboard connected to the LifeWindow and the Windows NT Security screen will appear. Click in “**Task Manager**”.
7. In Task Manager Click on “**New Task**”.
8. Go to the arrow situated in the right corner of the “**Open**” window and select “**C:\winnt\Explorer.exe**” and click in “**OK**”.
9. With the Explorer open, click in the “**Network Neighborhood**”, and then “**Entire Network**”, and then “**Microsoft Windows Network**”. The icon for the LifeWindow should appear and also the icons for all other computers in the network. The icon for the LifeWindow should be LW\_(serial number), for example LW\_V954349 for the LifeWindow serial number V954349.

10. Click in the computer that contains the LifeWindow files, go to the **“Folder”** that contains the LifeWindow update file (**LW6000 [Model] [Version]. exe**). Copy this file to the root directory of the LifeWindow drive C (**Dr\_C\_Vs/n**). Now on the left side of the Explorer, select the drive C so its contents are displayed in the right side. Looking in the right side, find the file **LW6000 [Model] [Version]. exe** and double click it. A window PKSFX will open. Just click on **“Extract”**. Another window will ask if you want to overwrite the files, click on **“Always”**. After updating, click once in the update file (**LW6000 [Model] [Version]. exe**) and then click in **“File”** and **“Delete”**. Confirm that you want to delete the file. The update zip file should be deleted to not occupy space in the LifeWindow flash memory.
11. After all files have been updated, double click the file **“LifeWindow”** that has a red ECG Icon. The LifeWindow will boot the new updated software. When the LifeWindow boots, you can click on **“System”** and then on **“About”** and confirm that the software is the correct updated version.