


## Digicare Biomedical Technology Inc

### Water Trap Instructions of Use for LW6000 with Gas Module

#### Intended Use

- Disposable water trap for the separation of water from the measuring line of monitors.
- To protect the monitor from water, bacteria and viruses.
- For use in the LW6000 with Anesthetics Gas module.
- Do not use the water trap with nebulizers.

#### Preparation

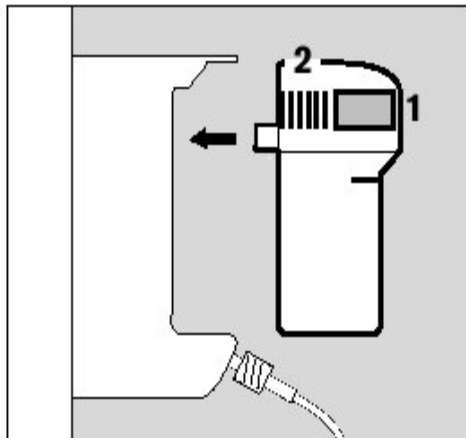


- Remove the new water trap from its packaging.
- Write the date in the area provided on the water trap.
- Hold the water trap by the ridged surfaces and insert it in the holder until you feel it click into place.

## Operation

- When the water has reached the full level mark, or if an occlusion message is displayed, remove the water trap. Grip it by the ridged surfaces and pull it out.

- **Draining the water trap:**

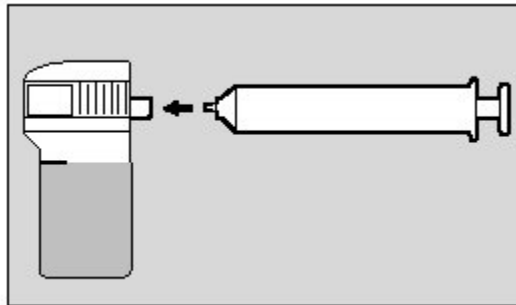

- Insert a 20 ml empty syringe without needle into the connection port.
- Draw off the water. Remove the full syringe and of it as normal hospital waste.
- Push the water trap back into the holder until it audibly clicks into place.

- **Replace the water trap:**

- If the occlusion message is not cancelled by draining the water trap.
- After the max. service life of four ( 4 ) weeks.
- If it is severely soiled.

- **Caution:**

- Do not allow any alcohol or cleaning agent / disinfectant to enter the water trap.
- Do not wash or sterilize the water trap. It may be damaged.
- Dispose of used water traps according to the regulation applicable.